

Think *Inside* The Box

The UV Box

If It Fits, It's Disinfected
in Less Than a Minute!

Hospitals invest \$100,000 or more in UV robots, foggers and an unending supply of wipes, but spend little to no money addressing the worst culprits of cross contamination: The mobile devices that move freely in, out and around your hospital.

Safety Net's AUVS KR615 – **The UV Box** – uses our Pentagon-selected **Photon Trap Technology** to disinfect these devices in less than a minute! The UV Box protects staff, patients and their families.

The UV Box:

- Proven Efficacy
- FDA Consulting Lab Tested
- Easy to Use
- Mobile & Accessible
- Durable
- Environmentally Friendly
- Affordable



Our **Photon Trap Technology**, used in The UV Box, is currently protecting the Pentagon from bioterrorism



"I love The UV Box and so does the staff and parents in NICU. We use it for cell phones, stethoscopes, thermometers, and EKG leads. We have also started using it for our name badges. My wish is to have one at every nurses station."

Unsolicited Hospital Testimonial

AUVS
ADVANCED ULTRA-VIOLET SYSTEMS

Available Exclusively Through Safety Net
Sales and Service 866-918-8546
info@safetynetamerica.com
www.safetynetamerica.com

Safety Net
Solutions that save lives.

The UV Box: Operating Information and Warranty

No Learning Curve!

- 1) Clean visible dirt, moisture or liquid residue from items to be disinfected as per manufacturer's instructions.
- 2) Place item in the KR615 Germicidal Enclosure. For items with cords (or leads) that must remain connected, thread the cord through the unit's Wire Routing System.
- 3) When disinfecting multiple items, do not stack or allow sides of items to touch.
- 4) Close the door and turn knob to lock. Press the red button to begin the decontamination cycle.
- 5) When the red button and LED indicator turn off, remove item(s) and use normally.



Place any item inside The UV Box without stacking or overlapping



Wire Routing System accommodates items that must remain connected



Close the door and turn knob to lock



Press the red button to begin the disinfection cycle

Note: Covered surfaces not contacted by the UV light will not be disinfected

Warranty

The KR615 UV disinfection Medium and Large Boxes are warranted for one (1) year from the date of delivery to the customer's facility. All parts and labor are covered under this period, excluding damage due to abuse by user.

LARGE BOX SPECIFICATIONS	
Overall Dimensions	10" H x 10" D x 20" W
Enclosure Dimensions	3.5" H x 8" D x 15.5" W
Weight	10 lbs.
Power	110V, AC
Bulb Life	16,000 hours = 50+ years under normal use

"We use it for glucometers, staff portable phones, cell phones, ID's, stethoscopes, telemetry units... The longer it is here the more people use it."
Unsolicited Hospital Testimonial



Available Exclusively Through Safety Net
Sales and Service Call: Indy Medical Supplies
indymed@gmail.com
866-744-9013

